Urias[®] Inflatable Splints

Hand Pump / Manometer Instructions for Use

Inflate the 'Urias[®]' Air Splint in the usual manner as detailed in the instructions for use, i.e. by opening the 'T' valve, inflating by mouth to the desired or recommended pressure, and then closing the 'T' valve.

Ensure that the Urias[®] Splint is inflated to the recommended / desired level by using the Hand Pump / Manometer.

- 1 Place the connector of the pump into the splint inlet tube
- 2 Ensure that the 'Orange' air release lock on the pump is in the closed position (clockwise)
- 3 Open the 'T' valve on the Urias[®] splint and read the pressure from the Manometer dial.
- 4 If additional pressure is required, use the Urias[®] hand pump to top up the pressure to the required level. Reading off dial and close the 'T' valve.
- 5 If less pressure is required, open the 'Orange' air release lock (anti-clockwise) on the pump and release air pressure to the required level reading on the dial and close 'T' valve of the Urias[®] splint.

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Arden Medical

CE
Geliefert von:
Arden Medical Ltd, Arden Road, Arden Forest Industrial Estate,
Alcester, Warwickshire B49 6HN, England

www.arden-medical.com